





# Flexima®/Softima® Active

---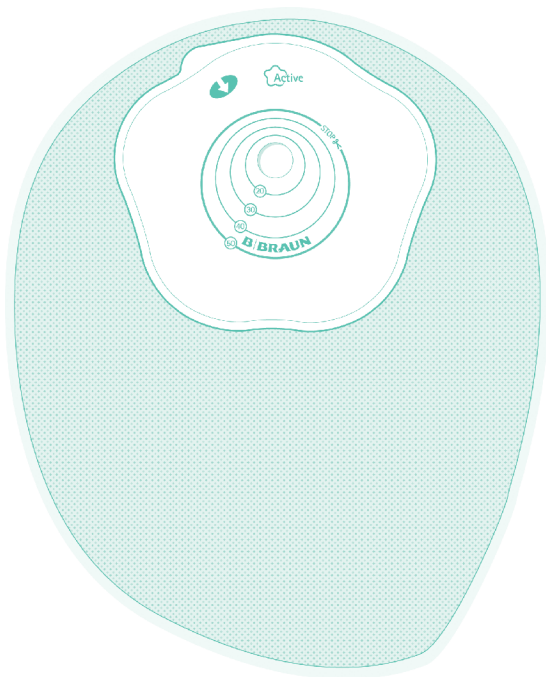







---

# Flexima®/Softima® Active

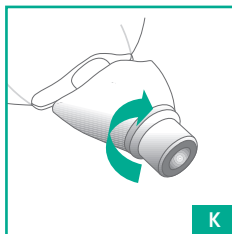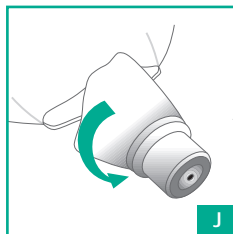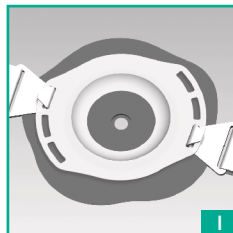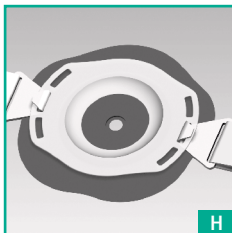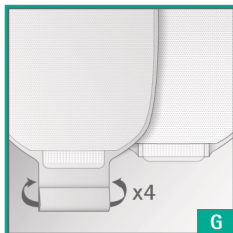---

# Flexima®/Softima® Active
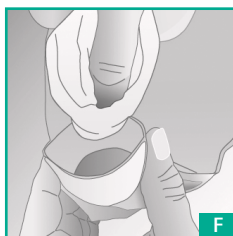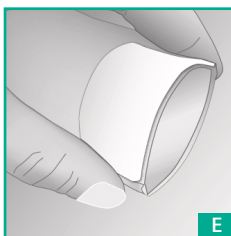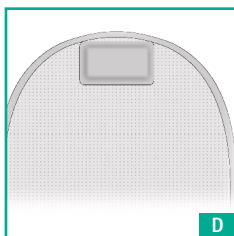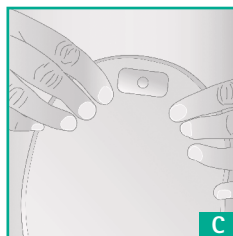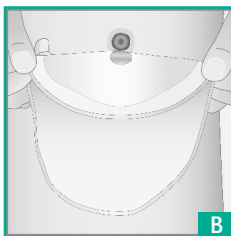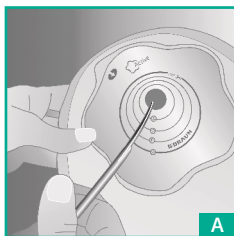
### What Flexima® / Softima® Active is and what it is used for

Flexima® / Softima® Active is a 1-Piece collection system for enterostomates or urostomates comprising of different types of pouches\* including a coupling ring with guiding system for easy pouch application:

- Closed for colostomates,
  - Drainable for enterostomates with either outlet,
  - Drainable for urostomates with a "Uro" tap and uro connector allowing connection to a B. Braun distal bag.
- Flexima® / Softima® Active may be used concomitantly with B. Braun stoma care accessories.

Intended purpose and indication:

The product is indicated for collection and containment of stools or urine by patients with enterostomy or urostomy. The product is not intended to be used on a skin or mucus membrane presenting a pathological change or change following disease or a wound.

Closed pouches are primarily intended for colostomates; drainable "Roll'Up" pouches for colostomates with soft or liquid stools and ileostomates; and drainable "Uro" pouches for urostomates.

The Soft Convex skin protector Flexima-Softima Active O'Convex is specifically designed to ensure a secure seal around stomas which are

- located in shallow to moderately deep skin folds,
- flush or slightly retracted.

The soft convex skin protector helps to prevent leakages, by tight-fitting the peristomal area. For extra security you can use one out of the three positions (H,I) of the belt loops of the soft convex skin protector to fix the belt.

#### Clinical benefits & performances:

Ostomy appliances compensate for the loss of faecal and/or urinary continence.

#### Patient population:

Enterostomates and urostomates adults or children above

four years.

#### Contraindication:

There are no formal contraindications for the use of Flexima® / Softima® Active.

#### Residual risks or side effects:

Peristomal skin complications can occur while using Flexima® / Softima® Active.

#### **1** Preparing the skin

Wash your hands before starting the procedure. Gently clean the peristomal area with lukewarm water. Avoid ether, alcohol or any other topical product which could interfere with base plate adhesion. Carefully dab dry the skin with soft and non-shedding cloths.

#### **2** Preparing the base plate

If you are using a Cut-To-Fit product, cut the central opening of the hydrocolloid adhesive to the size of your stoma (A) possibly with the help of the Stoma Guide. Do not cut beyond the maximum possible diameter and make sure that there is always a 2 to 3 mm margin around your stoma in order not to strangle or hurt the stomal mucosa. Note : Pre-Cut products are not intended to be cut out. In order to improve the adhesion of the Flexima-Softima Active O'Convex "Roll'Up" and the Flexima-Softima Active O'Convex Uro place hands over the base plate to warm it up.

#### **3** Fitting your appliance

Peel off the release. Position the bottom of the skin protector around the base of the stoma (B).

Press the bottom part of the skin protector smoothly onto the skin, then place and press onto the upper part (the skin protector is flexible and adapts perfectly to body movements) (C).

In case of error in the positioning, it is possible to remove the pouch and replace it properly. If you wish to use a belt (with a convex product), choose two loops that are diametrically opposed (H,I).

---

**MD**

- EN** Medical device as per 2017-745/EU
- DE** Medizinprodukt gemäß 2017-745/EU
- ES** Producto sanitario según el Reglamento (UE) 2017-745
- FR** Dispositif médical selon 2017-745/EU
- IT** Dispositivo medico secondo 2017-745/UE
- NL** Medisch hulpmiddel conform 2017-745/EU
- BG** Медицинско изделие съгласно 2017-745/EU
- CS** Zdravotnický prostředek dle nařízení (EU) 2017/745
- DA** Medicinsk udstyr i henhold til 2017-745/EU
- FI** Lääkinnällinen laite direktiivin 2017-745/EU mukaisesti
- EL** Ιατροτεχνολογικό προϊόν σύμφωνα με τον Κανονισμό 2017/745/ΕΕ
- ET** Meditsiinisead 2017-745/EU kohaselt
- HU** Orvosi eszköz, a 2017/745/EU rendeletnek megfelelően
- JA** 2017-745/EUに従った医療機器
- LV** Medicīniskā ierīce saskaņā ar 2017-745/ES
- PL** Wyrób medyczny zgodnie z rozporządzeniem (UE) 2017/745
- PT** Dispositivo médico de acordo com a diretiva 2017-745/EU
- RO** Dispozitiv medical conform 2017-745/EU
- RU** Медицинское изделие согласно 2017-745/EU
- SK** Zdravotnícka pomôcka podľa normy 2017-745/EU
- SL** Medicinski pripomoček v skladu z 2017-745/EU
- SR** Medicinsko sredstvo u skladu sa 2017-745/EU
- SV** Medicinteknisk produkt enligt 2017-745/EU

## EN Instructions for Use

### 4 Applying the pouch (closed & drainable)

For drainable "Roll'Up" close the closure outlet. For drainable "Uro" pouches, make sure that the tap is closed.

Separate the pouch walls by crumpling it slightly to introduce air into it.

Fold the extremity of the sleeve four times using the guide and fold the tab back.

To seal the outlet, apply pressure on all the length of the two self-fastening strips facing each other.

Please note: the closed sleeve is hidden under the soft non-woven fleece (G).

### 5 Handling drainable pouches

#### ■ Drainable "Roll'Up" pouches:

Closing the Roll'Up outlet:

Fold the extremity of the sleeve four times using the guides and fold the tab back. To seal the outlet, make sure that the two self-fastening strips are aligned, facing each other and apply pressure on all their length (G).

Emptying the pouch:

Position yourself appropriately, then release the two self-fastening strips and unroll the sleeve. Be careful, emptying will start right at this point. To fully open the outlet, apply slight pressure with your fingers on each side (E). Once the drainage is complete, expel, as much as possible, any matter still in the sleeve by pressing it lightly from top to bottom. Carefully clean its extremity (use only non-shedding material) (F). Finally, close the outlet again.

To close the pouch, fold the sleeve as indicated above.

#### ■ Drainable "Uro" pouches:

Closing the Uro tap:

To close the tap once the pouch has been emptied, turn clockwise until it has returned to its original position (J).

Emptying the pouch:

Point the tap towards the toilet bowl, slowly turn the tap counter-clockwise until the desired flow rate is obtained (K).

Connecting to B. Braun distal bag:

The connection of the pouch to a B. Braun distal bag can require the use of the supplied uro connector. To clip on the uro connector or the B. Braun Uribag, press the two wings and insert it into the pouch tap. Open the tap turning it half a turn counter-clockwise to allow urine to flow in the B. Braun distal bag. To un-clip, close the tap of the pouch by turning it half clockwise, pull out the uro connector or the B. Braun Uribag by pressing the two wings.

### 6 Using filter protection stickers

Before bathing or showering, stick the filter protection sticker over the gas evacuation openings (D) in order to maintain filter performances. For Split cover, this must require to lift the comfort film and to stick the filter protection filter directly on the pouch films. After bath or shower, remove the filter protection stickers.

### 7 Removing the products

Removing the pouch:

Pull slightly on the top of the pouch and remove the pouch from the top to the bottom while maintaining gentle pressure on the skin around the stoma with your other hand.

*\* Availability may vary from one country to another.*

If you note any change on your stoma and/or peristomal skin, please contact your health care professional.

National regulations for the destruction of potentially infectious materials must be observed.

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**JP** B. BRAUN AESCULAP JAPAN CO.,LTD
**SV** B. Braun Medical AB
B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud | FRANCE